

Model 128i Patient Positioning

“Dance Floor”
with the Hinge
panel down to
support the
patient

Proper position of
patient and X-Cel unit
for DP (AP) view. X-
Cel unit at 15 degree
angle.

Small “Wing”
folded down

The X-Cel can be
rotated to the 30
degree position to
take a lateral
oblique view

Large “Wing”
folded down



4 views can be taken with only moving the head and having the patient take one step sideways



Right AP



Right Oblique




Left AP



Left Oblique

Model 128i Patient Positioning

“Dance Floor”
with Hinge
panel against
the Upright
support

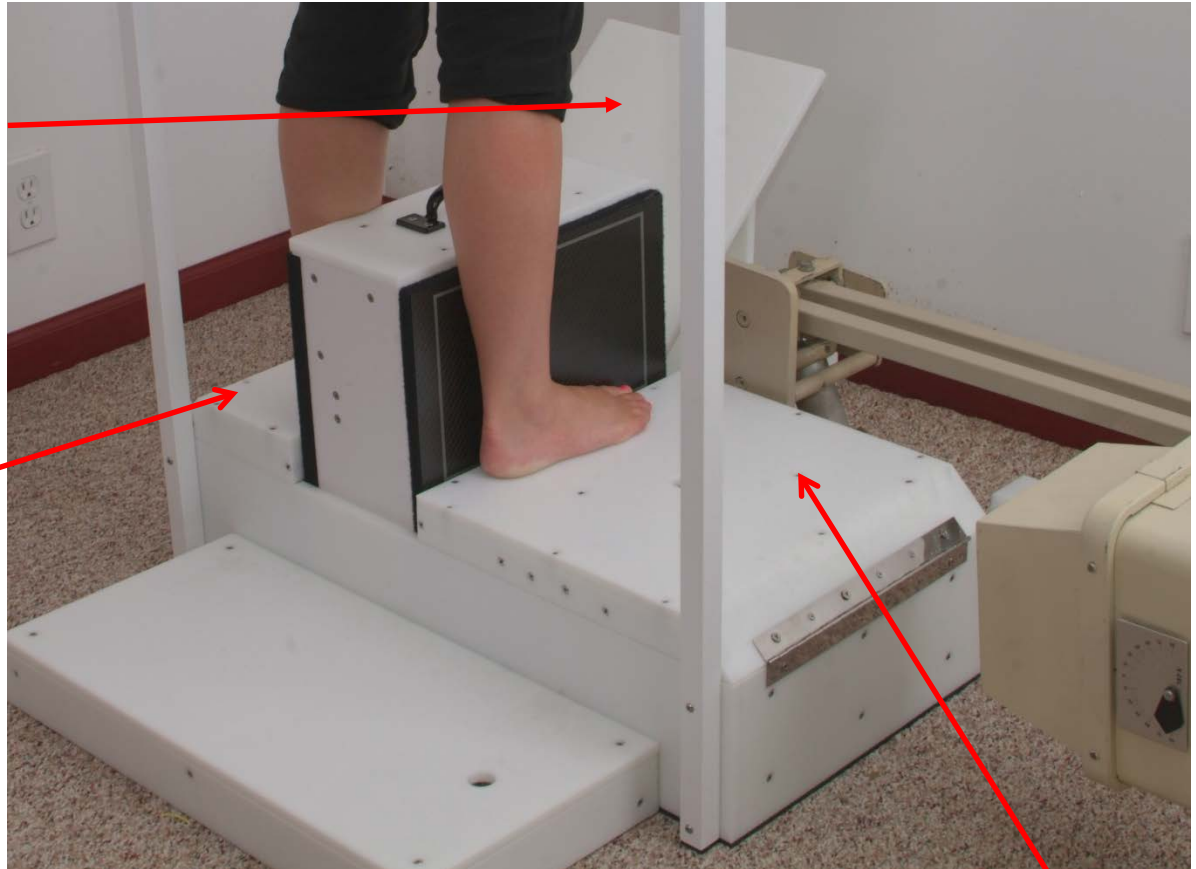

Small “Wing”

Large “Wing”

For a lateral of the left foot the patient will face away from the X-Cel

