



Ann Arbor Digital Devices, Inc.

19720 Cavanaugh Lake Rd.

Chelsea, MI 48118

(734) 730-2232 Fax (734) 475-3114


www.a2d2xray.com

a2d2help@gmail.com


Changing A2D2 Folder Permissions

The First step is to navigate to the “A2D2” folder in “Program Files (x86)”

Click on the Windows Flag
(this used to be the start button)

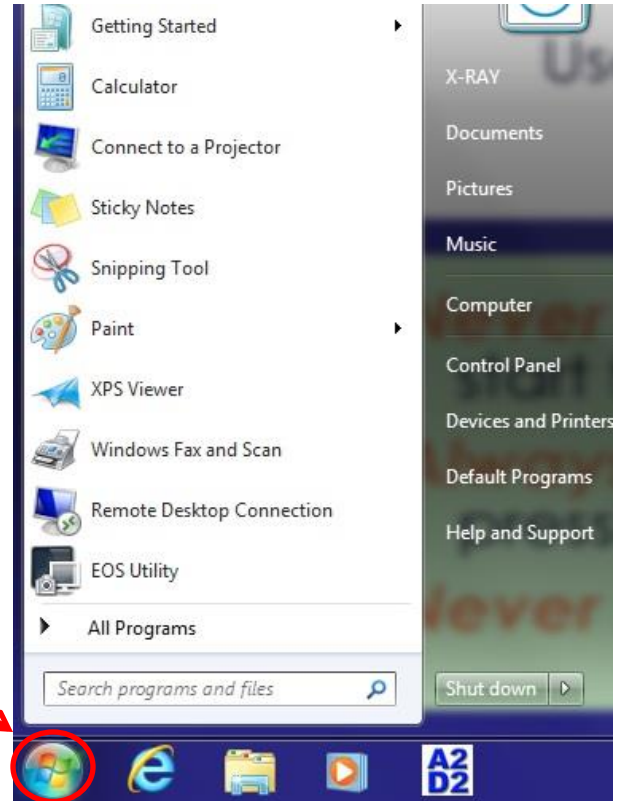


Click on “Computer”
(this used to be “My Computer”)

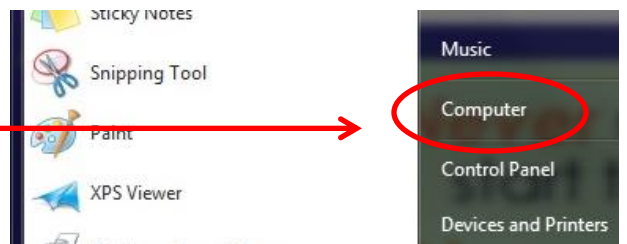


Double click on the “C” drive




Double click on "Program Files (x86)"

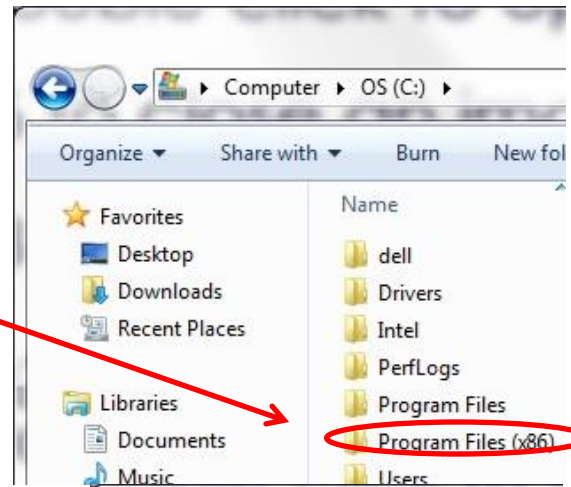


Single click on the "A2D2" folder to Highlight it

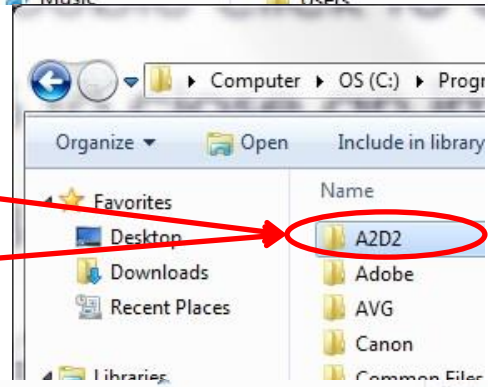

Right click on the "A2D2" folder

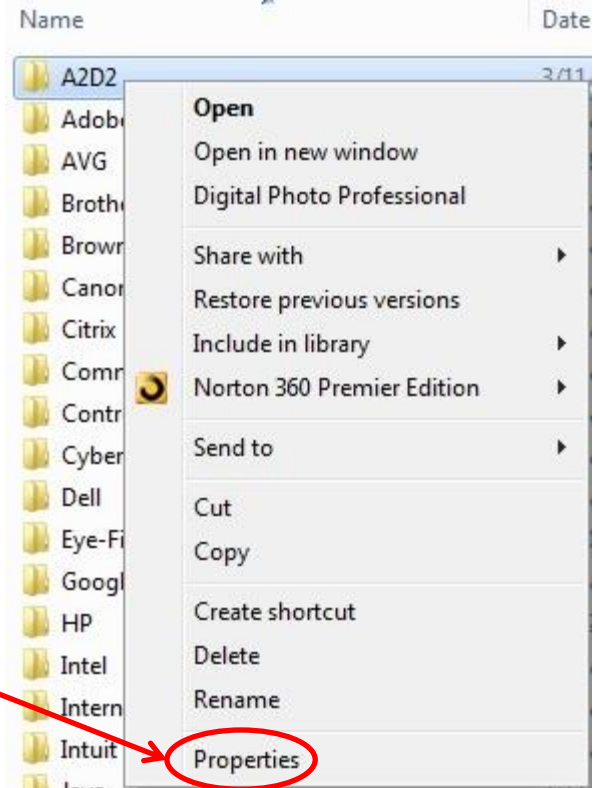


Select "Properties" from the pull down menu

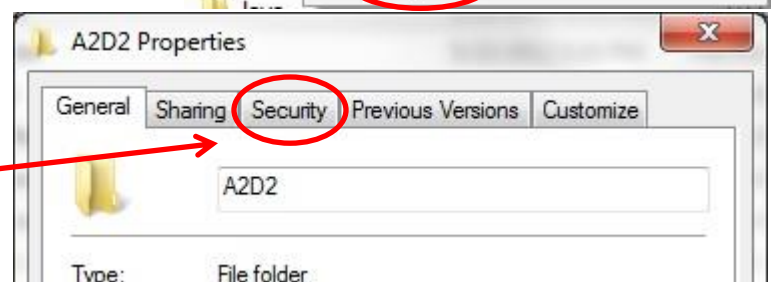


In the "Properties" window, click on the "Security" tab

To change permissions, click on the “Edit” button

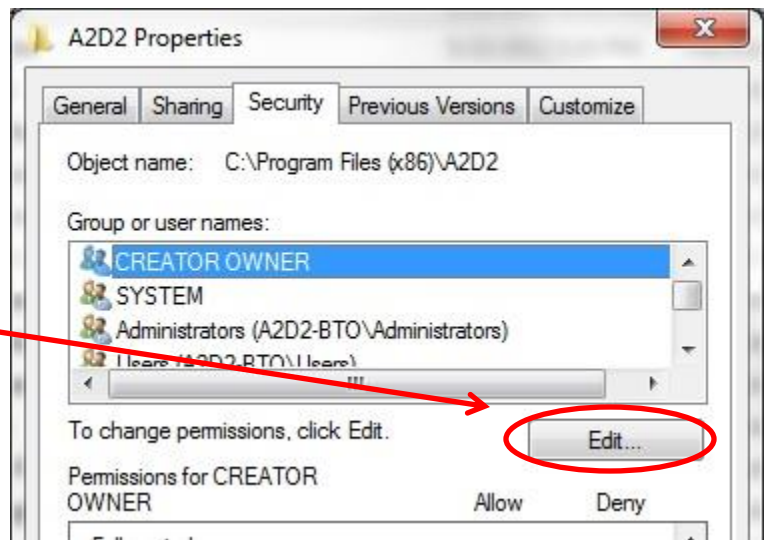

Click on “Users (X-RAY-PC\Users)” to highlight it

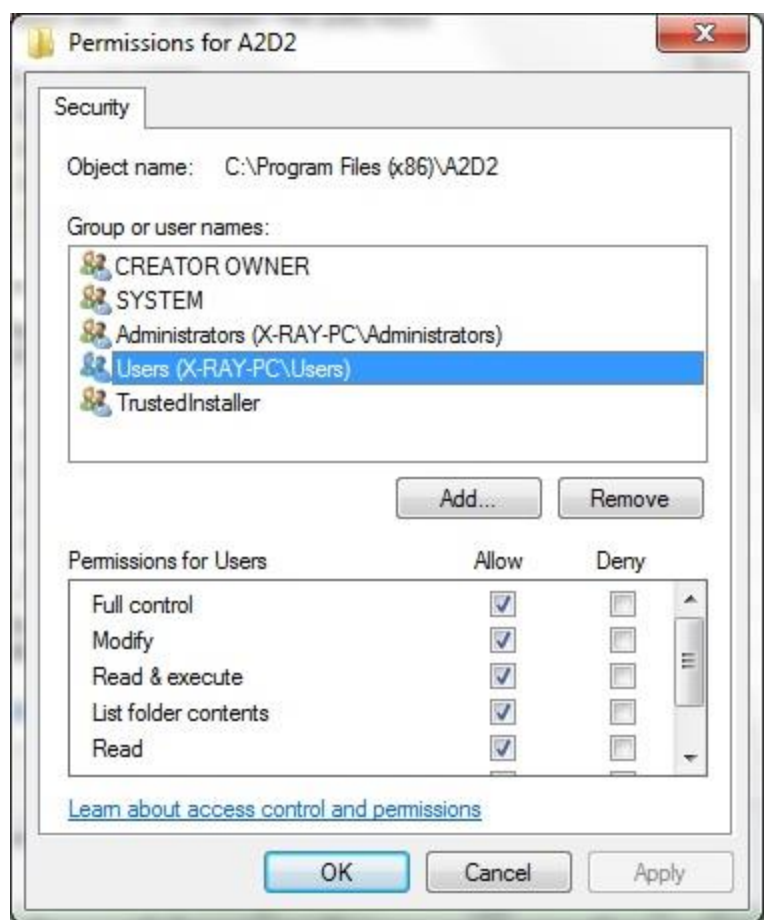

Click on the box under “Allow” and next to “Full Control”, this will put a checkmark in every box under “Allow” and allow all users of the computer to make changes to the contents of the A2D2 folder

Click on “OK”, this will apply the changes and close out the window

Now anyone using the X-RAY-PC can make changes to the A2D2 folder and it’s contents, most importantly the Display Factors File